


ZSI 475

HYDRAULIC PENILE IMPLANT

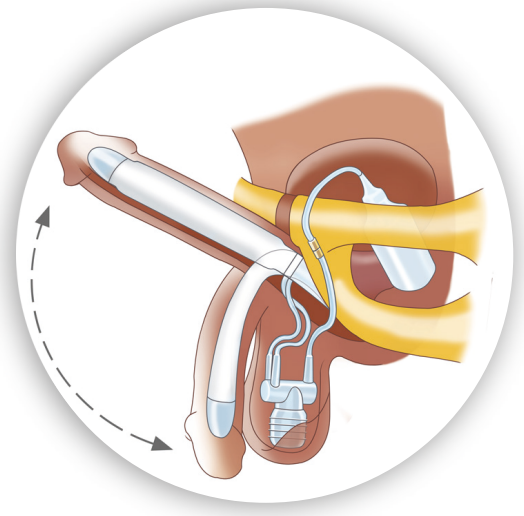

A solution to erectile dysfunction

- all standard sizes in one box.
- for more information, visit www.zsimplants.ch

ZSI is ISO 13485 : 2004 / ZSI 475 is EC certificated

Local Distributor

ZSI (Zephyr Surgical Implant Sàrl)
C/Ulmann, route des Jeunes 4bis
1227 Les Acacias - Switzerland


Flyer, English version
Version 3, Aug 2016




ALL STANDARD LENGTHS OF PENILE IMPLANT AND ACCESSORIES ARE AVAILABLE IN ONE BOX*

* Large Penis size part of 22 cm from 21 to 25 cm with extensions available in a separate box .



INTRODUCER WITH RULER

SPATULA

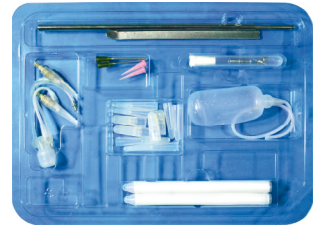

PROXIMAL PARTS

PUMP



NEEDLES FOR INTRODUCER AND CONNECTOR

RESERVOIR 80 ml



NEEDLES FOR SOLUTION FILLING



Length 13 16 19 (cm)
When Inflated Ø 16 16 18 (mm)

DISTAL PARTS CAVERNOUS CORPUS PROSTHESIS

Length 13cm and 16cm Ø RING 12,5mm. Length 19cm Ø RING 14mm.



cm 1 2 1 2 3
DISTAL AND PROXIMAL EXTENSIONS

ASSEMBLING



SELECT THE RIGHT DISTAL PART LENGTH



ADD EXTENSION IF NECESSARY




SCREW THE DISTAL PART TO THE PROXIMAL PART

Big Size Length 22 cm, Diameter 21 mm when inflated. Ø RING 14 mm.




PRESS THE BULB TO INFLATE THE PROSTHESIS
THE DIAMETER INCREASES AS THE PROSTHESIS IS INFLATED


FUNCTIONING

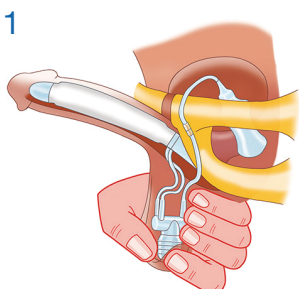


1 Press the bulb to inflate the prosthesis

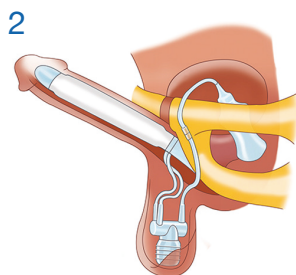

2 Penis erect

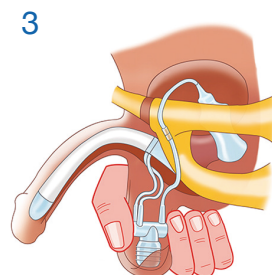

3 Press the cylinder to deflate the prosthesis

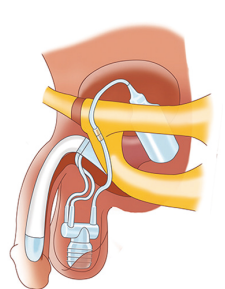

4 Penis flaccid